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Setting the fast movements duration after which the notification goes off	Switch # 1	Switch #2
10 seconds	ON (up)	ON (up)
13 seconds (default)	OFF (down)	OFF (down)
16 seconds	ON (up)	OFF (down)
20 seconds	OFF (down)	ON (up)

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Time delay	Switch # 3	Switch # 4	Switch #5
No Delay	OFF (down)	OFF (down)	OFF (down)
Disabled (default)	ON (up)	OFF (down)	OFF (down)
3min Delay	ON (up)	ON (up)	OFF (down)
6min Delay	OFF (down)	OFF (down)	ON (up)
10min Delay	ON (up)	OFF (down)	ON (up)
15min Delay	OFF (down)	ON (up)	ON (up)
30min Delay	ON (up)	ON (up)	ON (up)

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SW 1 switch function	Switch #6
The SW1 switch acts as the On/Off switch which means that the device can be switched on or off by pressing the switch for three (3) seconds.	OFF (down)
The SW1 switch does not act as the On/Off switch. The device is always on when it is connected to the power supply or when the batteries are in.	ON (up)

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Level	Switch #7	Switch #8
Very loud	OFF (down)	OFF (down)
Normal	ON (up)	OFF (down)
Quiet	OFF (down)	ON (up)
Mute	ON (up)	ON (up)

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### 13.1. Green – Presence

Light flashes every second time compared to blue light.	A person is on the bed or other movement is being made.
Light flashes the same speed as blue light.	A person has been on the bed for 60 seconds and the device has become activated.
Light flashes rapidly.	Fast-paced movement on the bed detected.
Light is off.	No one is on the bed.

### 13.2. Blue – Device on/standby

Light is off.	Device is switched off.
Light flashes slowly.	Device is switched on.
Light flashes rapidly a few times.	Device triggers a notification.

### 13.3. Red – Malfunction

Light is flashing the same speed as blue light	Bed sensor is disconnected or defective. Signal alarm sounds after 10 seconds and then every 45 seconds. There is also an alarm sent through the X2 connector after 30 seconds and then every 30 minutes.
Light is flashing every fourth time compared to blue light.	Batteries are empty. Replace batteries.

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### 14.1. Alarm signals

Two high and low "BEEPS" in a loop.	Fast movements alarm goes off.
Three high and low "BEEPS" in a loop.	Bed exit alarm goes off.

### 14.1. Information signals

Short, high "BEEP".	SW1 is pressed and device turns on.
Short, low "BEEP".	SW1 is pressed and device is turned off.
Short, low "BEEP" x 2.	SW1 has been pressed once and alarm signal is acknowledged.
Short, high "BEEP" x 3 and short low "BEEP" x 3.	Sensor is disconnected or broken. Information signal sounds after 10 seconds of disconnection and then every 45 seconds.
Short, high "BEEP", short "LOW" beep.	Batteries are empty. Replace batteries.
Short, high "BU-BEEP" x 3.	Sensitivity calibration has started / successful calibration.
Short, low "BEEP" (after calibration).	Sensitivity too low.
Short, low "BEEP" (after calibration) x 2.	Sensitivity too high.

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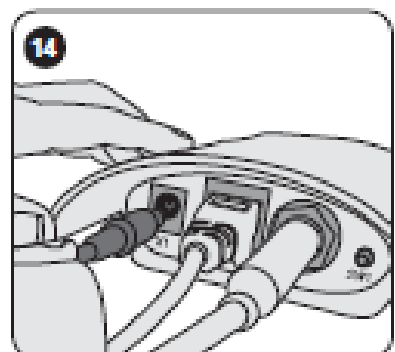
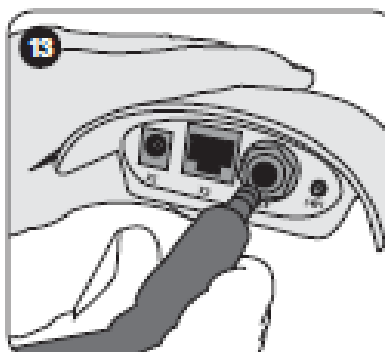
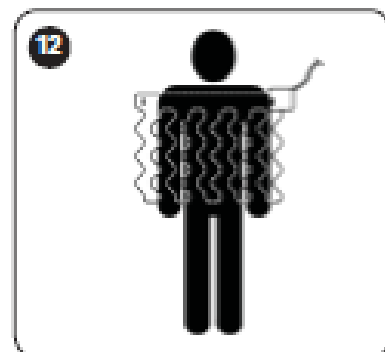
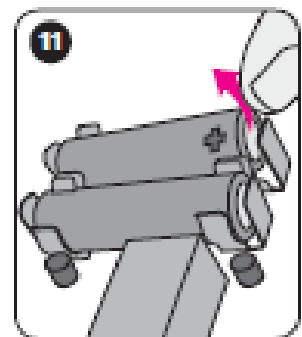
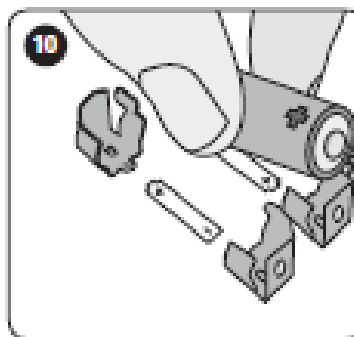
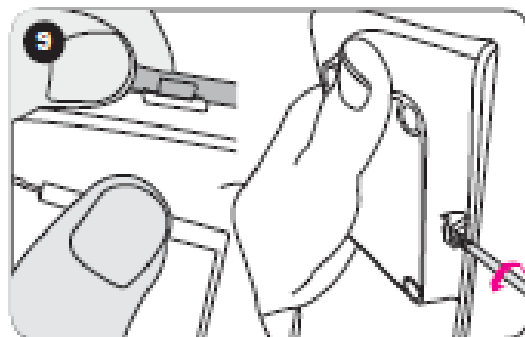
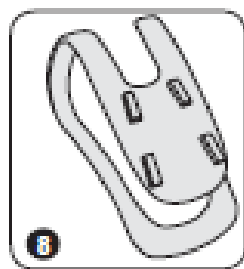
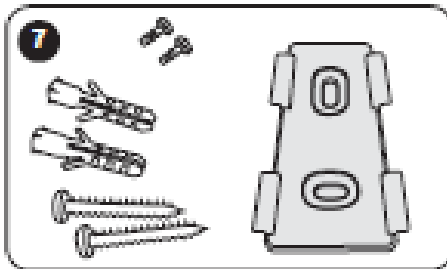
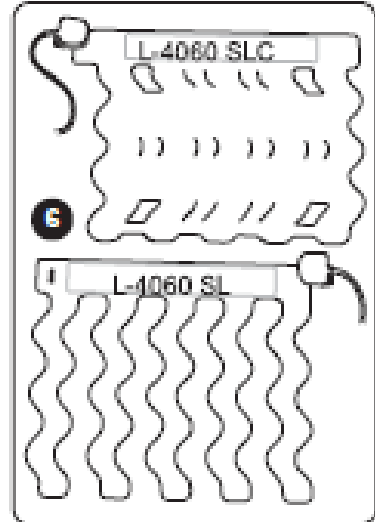
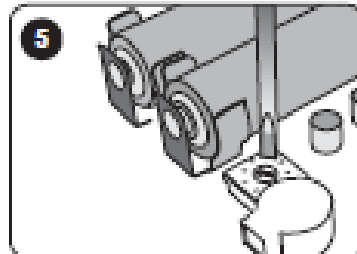
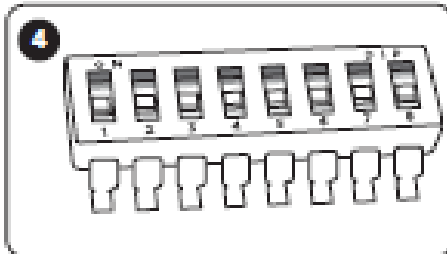
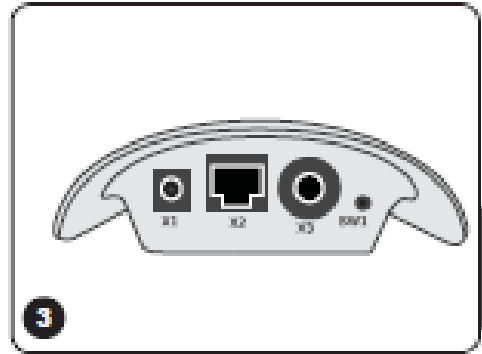
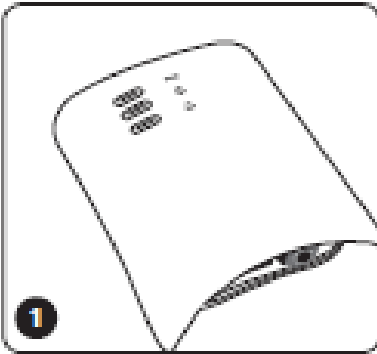
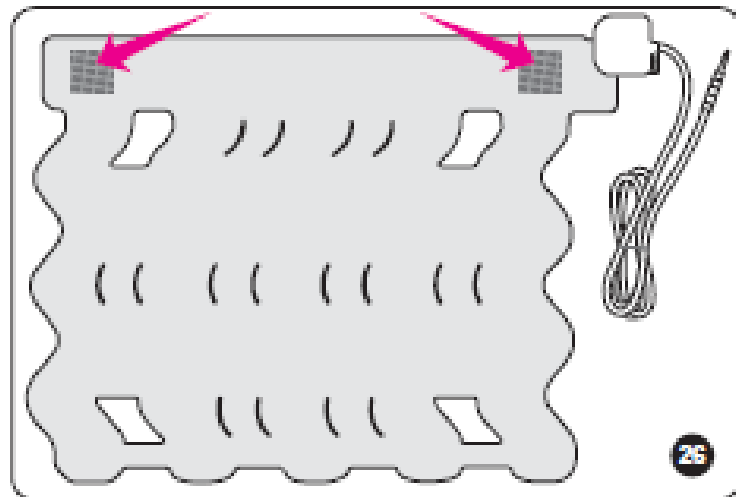
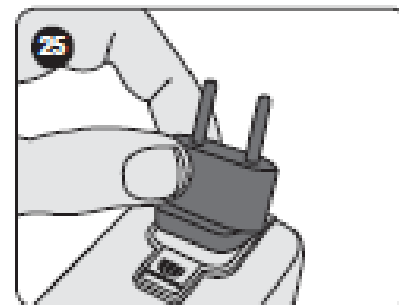
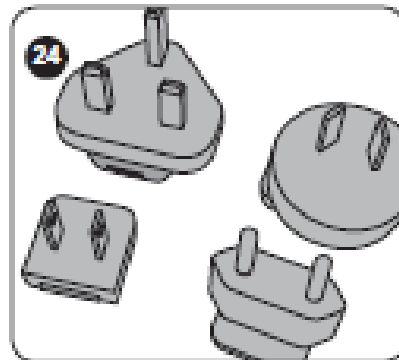
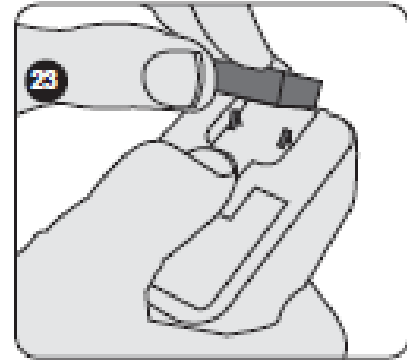
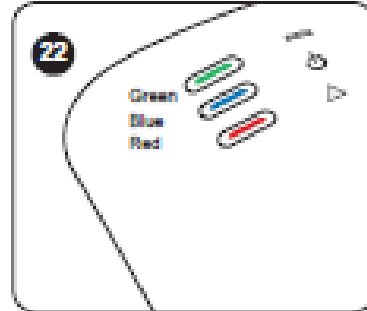
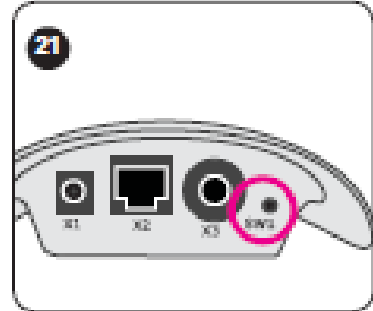
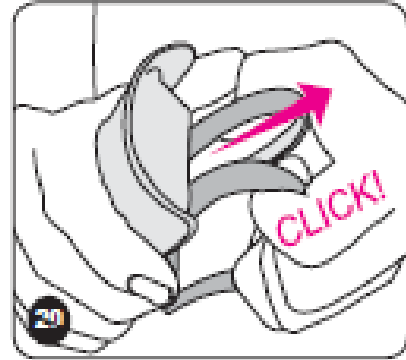
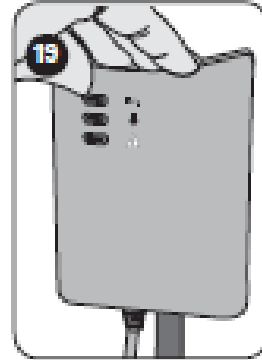
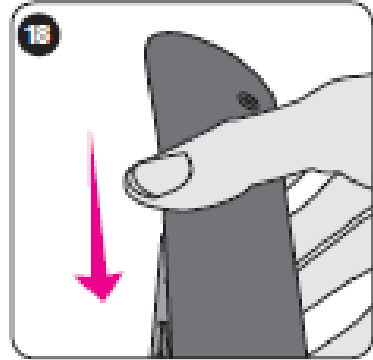
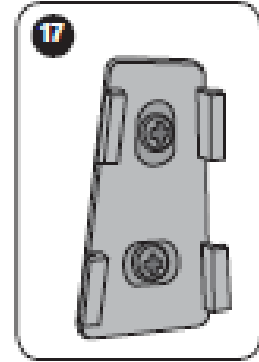
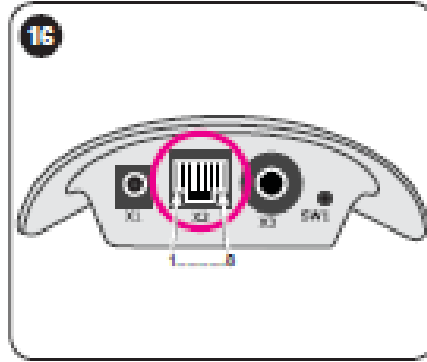
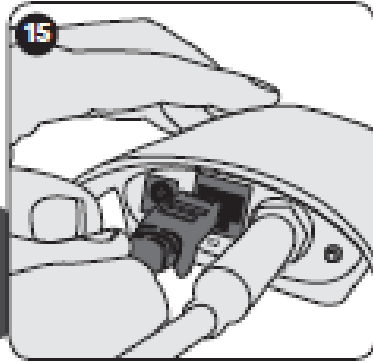
### Emfit Tonic-Clonic Seizure Monitor Troubleshooting Flow Chart